


Bischoff & Bischoff


Instruction manual

VFG
GS-R/L



Enjoy mobility.

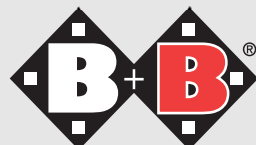

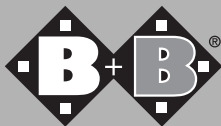

VFG (four-footed walking stick) GS- R/L (walking stick)

Dear customer,

This instruction manual is designed to familiarise you with the use and maintenance of your B+B walking aid and to assist in making your life safer.

We hope you enjoy our product and are more mobile with our product.

1. Adjusting height of stick

- Loosen retainer nut
- Press spring pin and set walking stick to the orthopaedically correct position.
- Allow spring pin to lock
- Tighten retainer nut

2. Caution - safety instructions - Risk analysis as per DIN EN ISO 13485

- Max. load capacity 100 kg
- Make sure that the connectors are correctly seated
- Instructions for adjusting the height:
- Make sure that the spring pin is correctly seated.
- Make sure that the retainer nut is correctly seated.
- Place on solid, horizontal surface only.

3. Technical data

	VFG	GS-R/L
Weight	0,8 kg	0,3 kg
Height	74-97 cm	74-97 cm
Increments	10	10
Colour	Bronze anodised	Bronze anodised

Warranty certificate

(Please note our general business terms & conditions)

Typ: VFG / GS-R/L

Serial number:*

(to be entered by dealer)

Dealer:

Date and stamp

* See the side frame for information stickers

Bischoff & Bischoff GmbH

Becker-Goering-Strasse 13 • D-76307 Karlsbad

Tel.: +49 (0)700/6000 7070 • Fax: +49 (0)700/6000 8080

www.bischoff-bischoff.com

info@bischoff-bischoff.de

